


PRODUCT structure

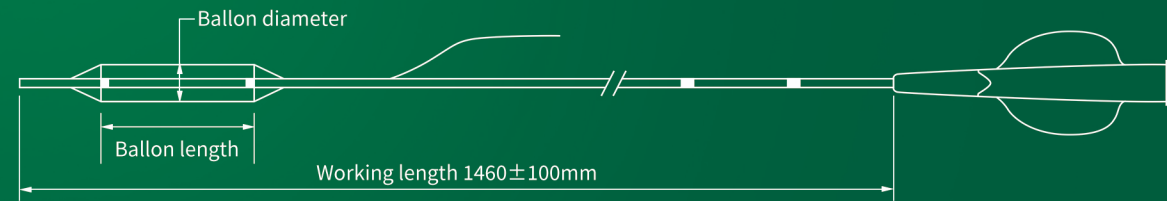

PRODUCT parameters

Specification and model:

Balloon Length /mm	Balloon diameter /mm									
	1.50	2.00	2.25	2.50	2.75	3.00	3.25	3.50	4.00	4.50
6	✓	✓	✓	✓						
9	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
12	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
15	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
20	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
25						✓	✓	✓	✓	✓

Intracranial balloon dilation catheter recommended inflation pressure


Nominal pressure NP ■ Rated bursting pressure RBP*

Dilation pressure atm	Balloon diameter /mm									
	1.50	2.00	2.25	2.50	2.75	3.00	3.25	3.50	4.00	4.50
1.0/101	1.15	1.62	1.85	2.07	2.29	2.51	2.76	2.99	3.46	3.91
2.0/203	1.30	1.78	2.01	2.24	2.47	2.70	2.95	3.19	3.66	4.12
3.0/304	1.38	1.86	2.10	2.33	2.56	2.81	3.06	3.30	3.78	4.25
4.0/405	1.43	1.92	2.16	2.41	2.65	2.89	3.14	3.39	3.87	4.35
5.0/507	1.47	1.97	2.21	2.46	2.71	2.95	3.20	3.45	3.94	4.43
6.0/608	1.50	2.00	2.25	2.50	2.75	3.00	3.25	3.50	4.00	4.50
7.0/709	1.52	2.02	2.27	2.53	2.78	3.04	3.29	3.59	4.05	4.56
8.0/811	1.54	2.04	2.29	2.55	2.81	3.07	3.32	3.63	4.10	4.62
9.0/912	1.56	2.06	2.31	2.58	2.84	3.10	3.36	3.67	4.15	4.69
10.0/1013	1.57	2.08	2.34	2.60	2.87	3.13	3.40	3.71	4.21	4.75
11.0/1115	1.59	2.10	2.36	2.62	2.90	3.16	3.44	3.75	4.27	4.81
12.0/1216	1.60	2.12	2.38	2.65	2.93	3.19	3.48	3.79	4.33*	4.88*
13.0/1317	1.62	2.15	2.41	2.68	2.96	3.23	3.52	3.83	4.39	4.96
14.0/1419	1.64*	2.17*	2.44*	2.71*	2.99*	3.26*	3.56*	3.87*	4.45	5.03
15.0/1520	1.66	2.20	2.47	2.74	3.02	3.29	3.60	3.91	4.52	5.12
16.0/1621	1.68	2.23	2.50	2.77	3.05	3.33	3.64	3.95	4.59	5.21

* This product is suitable for the interventional treatment of symptomatic intracranial atherosclerotic stenosis in non-acute phase. It can improve the blood perfusion of intracranial artery by balloon dilatation. Intracranial arteries included: basilar artery, middle cerebral artery, intracranial carotid artery and intracranial vertebral artery.

Zenith Intracranial Balloon Dilatation Catheter

NMPA No.20233030479

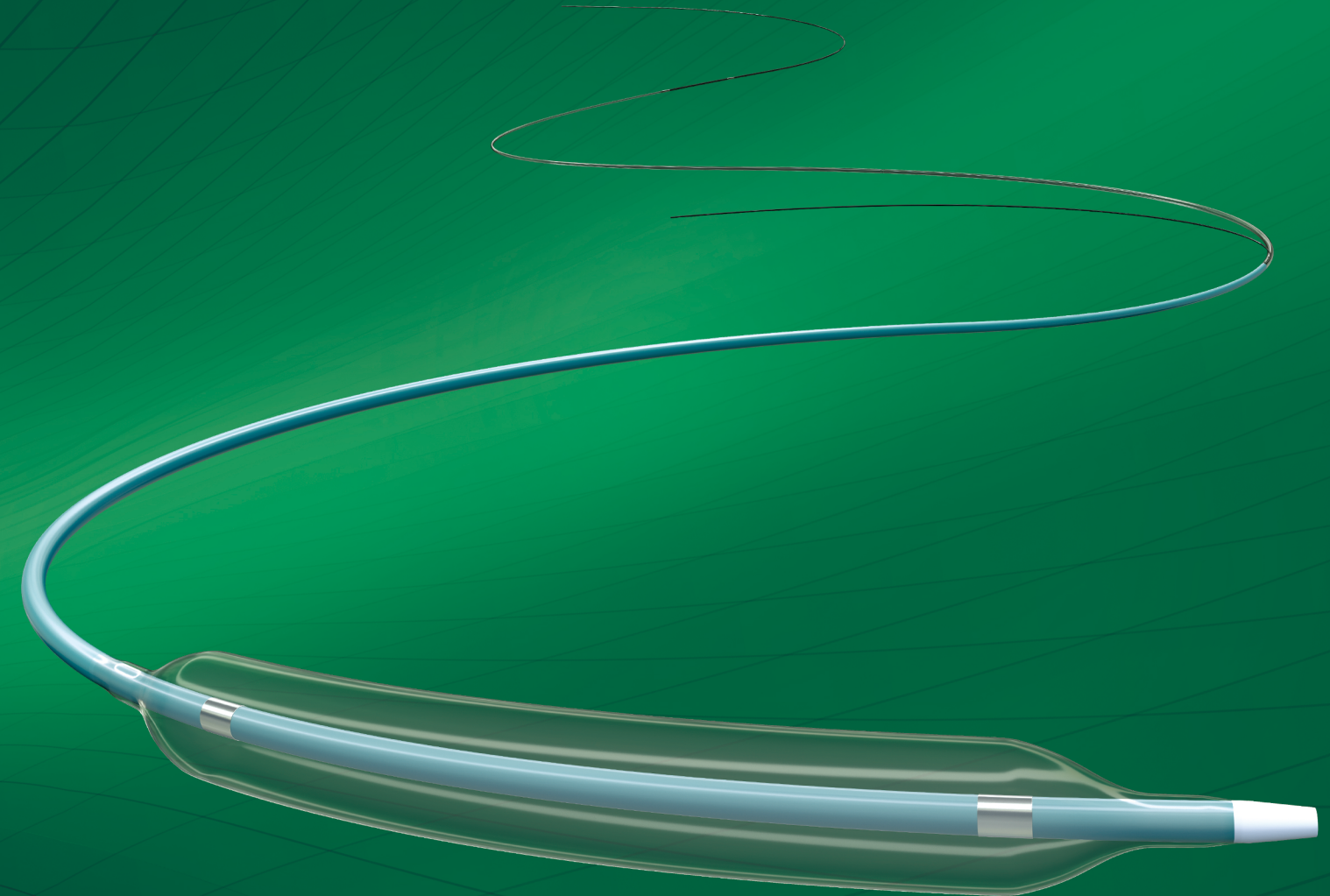

Zenith Vascular Scitech

TEL: 86-0512-6913873

E-mail: Information@zenithmedtech.com

Website: Zenithmedtech.com

DEVELOPING INNOVATION TECHNOLOGIES TO OPTIMIZE
VASCULAR TREATMENT

ZENITH BALLOON DILATATION CATHETER



01 250mm hydrophilic coating

02 0.0165inch low entry profile tip

03 Semi-compliant balloon

